

Sienna COVID-19 Antigen Rapid Test Kit

Results in 10 minutes using a Nasopharyngeal swab

Test Details

The Sienna COVID-19 Antigen Rapid Test Cassette is a rapid chromatographic immunoassay for the qualitative detection of COVID-19 antigen in Nasopharyngeal swab. The identification is based on the monoclonal antibodies specific for the Nucleocapsid protein of SARS-CoV-2. It is intended for professional in-vitro diagnostic use only.

Accurate Results

- Sensitivity: 95.0%
- Specificity: 99.2%
- Accuracy: 98.8%

Unmatched Convenience


- Visual Results: Easy to interpret.
- Fast Results: 10-minute test time.
- Individual Buffer Vials and Sterile Swabs:
Helps in testing multiple individuals simultaneously.



Sienna COVID-19 Antigen Rapid Test Kit

During the testing process, the extracted specimen reacts with the antibody of the SARS-CoV-2 nucleocapsid protein that are coated onto particles. The mixture migrates into the membrane to react with the antibody for the SARS-CoV-2 nucleocapsid protein on the membrane and will generate one colored line in the test regions. The presence of this colored line in the test regions indicates a positive result. A colored line will always appear in the control region to serve as a procedural control if the test has been performed correctly.

Test Procedure

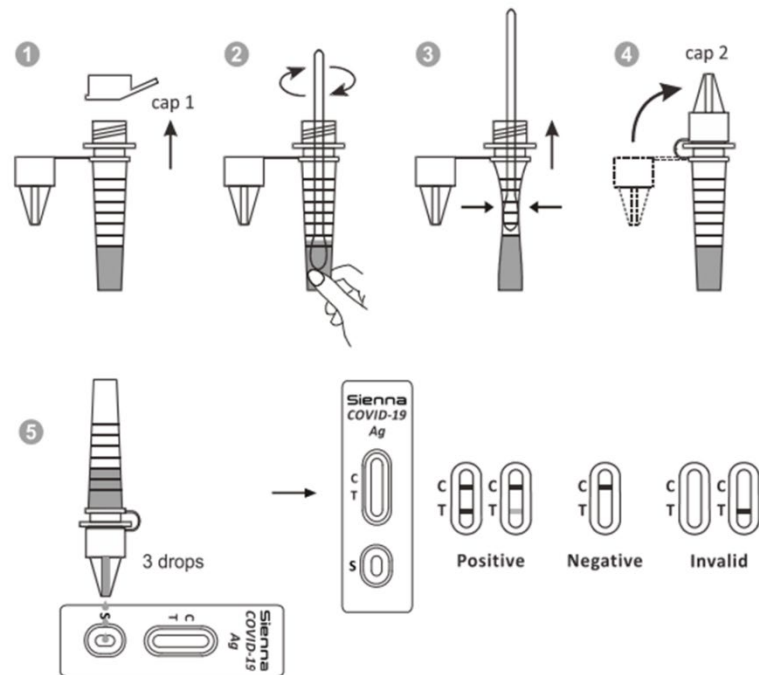

Kit Contains:

- (1) IFU
- (25) Individually pouched Cassettes
- (1) Workstation
- (25) Buffer Tubes
- (25) Sterile Swabs

Order and Shipping Information

Cat. No. 102242	Product: Sienna COVID-19 Antigen Rapid Test Contents: (1) IFU (25) Individually pouched Cassettes (1) Workstation (25) Buffer Tubes (25) Sterile Swabs	QTY 25 Tests	Registration Status CE EUA Authorized May 2021
---------------------------	---	------------------------	---

Item	Quantity	Length	Width	Height	Box Weight
Box	25 Tests	5 in	9.35 in	2.66 in	0.57 lbs
		12.25 cm	23.75 cm	6.75 cm	0.26 kg
Carton	49 Boxes (1,225 test)	24 in	16 in	20 in	46 lbs
		60 cm	40 cm	50 cm	20.8 kg
Pallet	8 Cartons	48 in	42 in	72 in	370 lbs
		120 cm	105 cm	180 cm	170 kg
NOTE: Total weight of Product Approx. (200kg) AND Pallet + other packaging (25kg) together = 200 kg Air Passenger Plane Cargo Only.					
Pallet	12 Cartons	48 in	42 in	72 in	550 lbs
		120 cm	105 cm	180 cm	250 kg
NOTE: Total weight of Product (265kg) AND Pallet (11kg) together = 265 kg Air Freight Cargo Only					
Pallet	16 Cartons	48 in	42 in	72 in	575 lbs
		120 cm	105 cm	180 cm	260 kg
NOTE: Total weight of Product Approx. (260kg) AND Pallet + other packaging (25kg) together = 300 kg Air Cargo Freight Only Short Height.					



For additional information
or to request a quote, email:
orders@seroclinix.com



Toll Free
1-877-453-0537

Authorized Distributor and Importer of record:

Seroclinix

CORPORATE OFFICE & LAB
5700 Timberlea Blvd, Unit 1
Mississauga Ontario Canada L4W 5B9

www.seroclinixhealthcare.com

US LABS
170 Northpointe Parkway, Suite 500
Amherst, NY USA 14228

US WAREHOUSE
1100 Military Road
Kenmore, NY 14217

DALLAS, TX
Summer 2021

EUROPE
The Netherlands
Fall 2021

Manufacturing Partner:



SALOFA OY
Örninkatu 15, 24100 Salo, Finland
www.salofa.com

Partner info