

December 22, 2021

Jack Feng iHealth Labs, Inc. 120 San Lucar Ct. Sunnyvale, CA 94086

Device: iHealth COVID-19 Antigen Rapid Test

EUA Number: EUA210470

Company: iHealth Labs, Inc.

Indication: Non-prescription home use for the qualitative detection of

nucleocapsid protein antigen from SARS-CoV-2 with:

Self-collected anterior nasal (nares) swab samples from individuals aged 15 years or older with symptoms of COVID-19 within the

first 7 days of symptom onset.

Adult-collected anterior nasal (nares) swab samples from individuals aged 2 years or older with symptoms of COVID-19

within the first 7 days of symptom onset.

Self-collected anterior nasal (nares) swab samples from individuals aged 15 years or older, or adult-collected anterior nasal (nares) swab samples from individuals aged 2 years or older, with or without symptoms or other epidemiological reasons to suspect COVID-19 when tested twice over three days with at least 24

hours (and no more than 48 hours) between tests.

Dear Mr. Feng:

On November 5, 2021, based on your<sup>1</sup> request the Food and Drug Administration (FDA) issued an Emergency Use Authorization (EUA) for emergency use of the iHealth COVID-19 Antigen Rapid Test, pursuant to Section 564 of the Federal Food, Drug, and Cosmetic Act (the Act) (21)

<sup>&</sup>lt;sup>1</sup> For ease of reference, this letter will use the term "you" and related terms to refer to iHealth Labs, Inc.

U.S.C. §360bbb-3) for the indication stated in the letter.<sup>2</sup> Based on your request, FDA reissued the letter in its entirety with revisions incorporated on December 3, 2021.<sup>3</sup>

On December 11, 2021, you requested to amend your EUA. Based on that request, and having concluded that revising the December 3, 2021, EUA is appropriate to protect the public health or safety under section 564(g)(2)(C) of the Act (21 U.S.C. § 360bbb-3(g)(2)(C)), FDA is reissuing the December 3, 2021, letter in its entirety with the revisions incorporated.<sup>4</sup> Pursuant to section 564 of the Act and the Scope of Authorization (Section II) and Conditions of Authorization (Section IV) of this reissued letter, your product<sup>5</sup> is now authorized for use consistent with the indication described above.

On February 4, 2020, pursuant to Section 564(b)(1)(C) of the Act, the Secretary of the Department of Health and Human Services (HHS) determined that there is a public health emergency that has a significant potential to affect national security or the health and security of United States citizens living abroad, and that involves the virus that causes COVID-19. Pursuant to Section 564 of the Act, and on the basis of such determination, the Secretary of HHS then declared that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of the virus that causes COVID-19 subject to the terms of any authorization issued under Section 564(a) of the Act.<sup>6</sup>

FDA considered the totality of scientific information available in authorizing the emergency use of your product for the indication above. A summary of the performance information FDA relied upon is included in the "iHealth COVID-19 Antigen Rapid Test Healthcare Provider Instructions for Use" identified below.

Having concluded that the criteria for issuance of this authorization under Section 564(c) of the

<sup>&</sup>lt;sup>2</sup> The November 5, 2021, letter authorized the iHealth COVID-19 Antigen Rapid Test for non-prescription home use for the qualitative detection of nucleocapsid protein antigen from SARS-CoV-2 with: (1) Self-collected anterior nasal (nares) swab samples from individuals aged 15 years or older with symptoms of COVID-19 within the first 7 days of symptom onset, (2) Adult-collected anterior nasal (nares) swab samples from individuals aged 2 years or older with symptoms of COVID-19 within the first 7 days of symptom onset and, (3) Self-collected anterior nasal (nares) swab samples from individuals aged 15 years or older, or adult-collected anterior nasal (nares) swab samples from individuals aged 2 years or older, with or without symptoms or other epidemiological reasons to suspect COVID-19 when tested twice over three days with at least 24 hours (and no more than 48 hours) between tests.

<sup>3</sup> On December 3, 2021, the revisions to the November 5, 2021, letter and authorized labeling included: (1) offer the test kit with 1-pack option, in addition to the 2-, 5- or 40-pack options currently offered, (2) add additional model numbers to further increase production capacity, (3) update Condition of Authorization S. to maintain the original 3 month timeline afforded in the November 5, 2021, letter, and (4) updates to the Fact Sheet for Healthcare Providers to fix a minor error and update the date to match the date of reissuance.

<sup>&</sup>lt;sup>4</sup> The revisions to the December 3, 2021, letter and authorized labeling include: (1) modification of the test kit (including end user "iHealth COVID-19 Antigen Rapid Test Instruction for use,") to offer two kit configurations: Test Set 1-includes a pre-filled Tube or Test Set 2 - empty Tube plus sealed Solution vial to further increase production capacity, (2) update Condition of Authorization S. below to maintain the original 3 month timeline afforded in the November 5, 2021, letter (and continued in December 3, 2021 letter) to 1.5 months, and (3) updates to the Fact Sheet for Healthcare Providers to fix a minor error and update the date to match the date of reissuance. <sup>5</sup> For ease of reference, this letter will use the term "your product" to refer to the iHealth COVID-19 Antigen Rapid Test for the indication identified above.

<sup>&</sup>lt;sup>6</sup> U.S. Department of Health and Human Services, *Determination of a Public Health Emergency and Declaration that Circumstances Exist Justifying Authorizations Pursuant to Section 564(b) of the Federal Food, Drug, and Cosmetic Act, 21 U.S.C.* § 360bbb-3. 85 FR 7316 (February 7, 2020).

Act are met, I am authorizing the emergency use of your product, described in the Scope of Authorization of this letter (Section II), subject to the terms of this authorization.

#### I. Criteria for Issuance of Authorization

I have concluded that the emergency use of your product meets the criteria for issuance of an authorization under Section 564(c) of the Act, because I have concluded that:

- 1. The SARS-CoV-2 can cause a serious or life-threatening disease or condition, including severe respiratory illness, to humans infected by this virus;
- 2. Based on the totality of scientific evidence available to FDA, it is reasonable to believe that your product may be effective in diagnosing COVID-19, and that the known and potential benefits of your product when used for diagnosing COVID-19, outweigh the known and potential risks of your product; and
- 3. There is no adequate, approved, and available alternative to the emergency use of your product. <sup>7</sup>

# II. Scope of Authorization

I have concluded, pursuant to Section 564(d)(1) of the Act, that the scope of this authorization is limited to the indication above.

### **Authorized Product Details**

Your product is a lateral flow assay intended for the qualitative detection of nucleocapsid protein antigen from SARS-CoV-2. This test is authorized for non-prescription home use with self-collected anterior nasal (nares) swab samples from individuals aged 15 years or older with symptoms of COVID-19 within the first 7 days of symptom onset. This test is also authorized for non-prescription home use with adult-collected anterior nasal (nares) swab samples from individuals aged 2 years or older with symptoms of COVID-19 within the first 7 days of symptom onset. This test is also authorized for non-prescription home use with self-collected anterior nasal (nares) swab samples from individuals aged 15 years or older, or adult-collected anterior nasal (nares) swab samples from individuals aged 2 years or older, with or without symptoms or other epidemiological reasons to suspect COVID-19 when tested twice over three days with at least 24 hours (and no more than 48 hours) between tests. Your product does not differentiate between SARS-CoV and SARS-CoV-2.

The SARS-CoV-2 nucleocapsid protein antigen is generally detectable in anterior nasal swab specimens during the acute phase of infection. Positive results indicate the presence of viral antigens, but clinical correlation with past medical history and other diagnostic information is necessary to determine infection status. Positive results do not rule out bacterial infection or coinfection with other viruses. The agent detected may not be the definite cause of disease. Individuals who test positive with your product should self-isolate and seek follow up care with

<sup>&</sup>lt;sup>7</sup> No other criteria of issuance have been prescribed by regulation under Section 564(c)(4) of the Act.

their physician or healthcare provider as additional testing may be necessary.

Negative results are presumptive and confirmation with a molecular assay, if necessary, for patient management, may be performed. Negative results do not rule out COVID-19 and should not be used as the sole basis for treatment or patient management decisions, including infection control decisions. Negative results should be considered in the context of an individual's recent exposures, history and the presence of clinical signs and symptoms consistent with COVID-19.

For serial testing programs, additional confirmatory testing with a molecular test for negative results may be necessary, if there is a high likelihood of SARS-CoV-2 infection, such as an individual with a close contact with COVID-19 or with suspected exposure to COVID-19 or in communities with high prevalence of infection. Additional confirmatory testing with a molecular test for positive results may also be necessary, if there is a low likelihood of SARS-CoV-2 infection, such as in individuals without known exposures to COVID-19 or residing in communities with low prevalence of infection.

Individuals who test negative and continue to experience COVID-19 like symptoms of fever, cough and/or shortness of breath may still have SARS-CoV-2 infection and should seek follow up care with their physician or healthcare provider.

Individuals should provide all results obtained with this product to their healthcare provider for public health reporting or by following the mobile application instructions for self-reporting. All healthcare providers will report all test results they receive from individuals who use the authorized product to relevant public health authorities in accordance with local, state, and federal requirements using appropriate LOINC and SNOMED codes, as defined by the Laboratory In Vitro Diagnostics (LIVD) Test Code Mapping for SARS-CoV-2 Tests provided by the Centers for Disease Control and Prevention (CDC).

Your product is performed using anterior nasal (nares) swab samples from individuals aged 2 years or older. The individual using your product is instructed to download, register and log into the mobile application (App) and follow the step-by-step based instructions on the iHealth COVID-19 Test App on a compatible smartphone. The individual will receive a kit with one of two possible configurations. When using your product, the individual first removes either: (1) Test Set 1 - the COVID-19 Test Card in Pouch, the Tube pre-filled with the extraction solution and the swab or (2) Test Set 2 - the COVID-19 Test Card in Pouch, empty Tube, sealed Solution and the swab. For Test Set 2, the individual proceeds to transfer the Solution into the empty Tube. For Test Set 1 and 2, the individual then opens the foil pouch containing COVID-19 Test Card. The swab is then removed from its packaging and the individual collects an anterior nasal swab sample by inserting the swab into the nostril and firmly and slowly brushing the insides of the nasal wall in a circular motion at least 5 times, taking at least 15 seconds to collect the specimen, before repeating the process in the second nostril. The swab is then immediately inserted into the tube and stir at least 15 times. The swab is then removed while pressing against the sides of the tube and the tube capped with the cap. The liquid in tube interacts with the

<sup>&</sup>lt;sup>8</sup> Compatible smartphone includes Apple iPhone running Operation System (iOS) 12 or later versions of the iOS, and Android Phones running Android 6.0 or later versions. Additional smartphone models as may be requested, and for which you receive appropriate authorization, in accordance with Condition L. below.

specimen and facilitates exposure of the appropriate viral antigens to the antibodies used in your product. Three drops of the solution are applied into the Sample Port of the COVID-19 Test Card. The individual then starts the 15 minute timer. If the extracted specimen contains SARS-CoV-2 antigens, a pink-to-purple T (Test) Line, along with a pink-to-purple C (Control) Line will appear on the COVID-19 Test Card indicating a positive result. This control line indicates that the sample has migrated across the membrane as intended and indicates that the test was correctly performed. Test results are interpreted visually after 15 minutes based on the presence or absence of visually detectable colored lines at the control line (C) and/or test line (T).

The iHealth COVID-19 Antigen Rapid Test includes the following materials or other authorized materials (as may be requested under Condition L. below): COVID-19 Test Card(s), Nasal Swab(s), the end user "iHealth COVID-19 Antigen Rapid Test Instruction for use," and either pre-filled Tube(s) (Test Set 1) or empty Tube(s) plus sealed Solution(s) (Test Set 2).

Your product includes an internal control test line ("C") that must generate the expected result for a test to be considered valid, as outlined in the "iHealth COVID-19 Antigen Rapid Test Instruction for use" and the "iHealth COVID-19 Antigen Rapid Test Healthcare Provider Instructions for Use."

The labeling entitled "iHealth COVID-19 Antigen Rapid Test Healthcare Provider Instructions for Use," the "iHealth COVID-19 Antigen Rapid Test Instruction for use," and the "iHealth COVID-19 Antigen Rapid Test" box labels (1-, 2-, 5- or 40-pack) (available at <a href="https://www.fda.gov/medical-devices/coronavirus-disease-2019-covid-19-emergency-use-authorizations-medical-devices/in-vitro-diagnostics-euas">https://www.fda.gov/medical-devices/coronavirus-disease-2019-covid-19-emergency-use-authorizations-medical-devices/in-vitro-diagnostics-euas</a>), the "iHealth COVID-19 Test" software App and the following fact sheet pertaining to the emergency use, is required to be made available as set forth in the Conditions of Authorization (Section IV), and are collectively referred to as "authorized labeling":

• Fact Sheet for Healthcare Providers<sup>9</sup>: iHealth Labs, Inc. - iHealth COVID-19 Antigen Rapid Test

The above described product, when accompanied by the authorized labeling as set forth in the Conditions of Authorization (Section IV) is authorized to be distributed and used under this EUA, despite the fact that it does not meet certain requirements otherwise required by applicable federal law.

I have concluded, pursuant to Section 564(d)(2) of the Act, that it is reasonable to believe that the known and potential benefits of your product, when used consistent with the Scope of Authorization of this letter (Section II), outweigh the known and potential risks of your product.

I have concluded, pursuant to Section 564(d)(3) of the Act, based on the totality of scientific evidence available to FDA, that it is reasonable to believe that your product may be effective in diagnosing COVID-19, when used consistent with the Scope of Authorization of this letter (Section II), pursuant to Section 564(c)(2)(A) of the Act.

<sup>&</sup>lt;sup>9</sup> Note that the information typically found in a Fact Sheet for Individuals is contained in the authorized "iHealth COVID-19 Antigen Rapid Test Instruction for use" that will be available to end users as set forth in the Conditions of Authorization (Section IV).

FDA has reviewed the scientific information available to FDA, including the information supporting the conclusions described in Section I above, and concludes that your product (as described in the Scope of Authorization of this letter (Section II)) meets the criteria set forth in Section 564(c) of the Act concerning safety and potential effectiveness.

The emergency use of your product under this EUA must be consistent with, and may not exceed, the terms of this letter, including the Scope of Authorization (Section II) and the Conditions of Authorization (Section IV). Subject to the terms of this EUA and under the circumstances set forth in the Secretary of HHS's determination under Section 564(b)(1)(C) of the Act described above and the Secretary of HHS's corresponding declaration under Section 564(b)(1) of the Act, your product is authorized for the indication above.

# III. Waiver of Certain Requirements

I am waiving the following requirements for your product during the duration of this EUA:

Current good manufacturing practice requirements, including the quality system requirements under 21 CFR Part 820 with respect to the design, manufacture, packaging, labeling, storage, and distribution of your product, but excluding Subpart H (Acceptance Activities, 21 CFR 820.80 and 21 CFR 820.86), Subpart I (Nonconforming Product, 21 CFR 820.90), and Subpart O (Statistical Techniques, 21 CFR 820.250).

### IV. Conditions of Authorization

Pursuant to Section 564(e) of the Act, I am establishing the following conditions on this authorization:

# iHealth Labs, Inc. (You) and Authorized Distributor(s)<sup>10</sup>

- A. Your product must comply with the following labeling requirements: the intended use statement in 21 CFR 809.10(a)(2), (b)(2); adequate directions for use in 21 U.S.C. 352(f) and 21 CFR 809.10(b)(5), (7), and (8); appropriate limitations on the use of the device including information required under 21 CFR 809.10(a)(4); and any available information regarding performance of the device, including requirements under 21 CFR 809.10(b)(12).
- B. You and authorized distributor(s) must make available the "iHealth COVID-19 Antigen Rapid Test Instruction for use" for your product in the shipped kit using the "iHealth COVID-19 Antigen Rapid Test" box labels (1-, 2-, 5- or 40-pack) and electronically on your website(s).

<sup>&</sup>lt;sup>10</sup> "Authorized Distributor(s)" are identified by you, iHealth Labs, Inc., in your EUA submission as an entity allowed to distribute the iHealth COVID-19 Antigen Rapid Test.

- C. You and authorized distributor(s) must maintain records of customer complaint files and report to FDA any significant complaints about usability or deviations from the established performance characteristics of which you and authorized distributor(s) become aware.
- D. You and authorized distributor(s) must inform relevant public health authorities of this EUA, including the terms and conditions herein, and any updates made to your product and/or the authorized labeling.
- E. Through a process of inventory control, you and authorized distributor(s) must maintain records of the locations (e.g., pharmacies, doctor's offices, etc.) to which your product is distributed and the number of tests distributed to each location.
- F. You and authorized distributor(s) must collect information on the performance of your product and have a process in place to track adverse events, including any occurrence of false positive or false negative results and significant deviations from the established performance characteristics of the product of which you become aware and report any such events to FDA in accordance with 21 CFR Part 803. Serious adverse events, especially unexpected biosafety concerns, should immediately be reported to the Division of Microbiology (DMD)/Office of Health Technology 7 (OHT7)-Office of In Vitro Diagnostics and Radiological Health (OIR)/Office of Product Evaluation and Quality (OPEQ)/Center for Devices and Radiological Health (CDRH) (via email: CDRH-EUAReporting@fda.hhs.gov).
- G. You and authorized distributor(s) are authorized to make available additional information relating to the emergency use of your product that is consistent with, and does not exceed, the terms of this letter of authorization.
- H. You and authorized distributor(s) using your product must ensure that any records associated with this EUA are maintained until otherwise notified by FDA. Such records will be made available to FDA for inspection upon request.

# iHealth Labs, Inc. (You)

- I. You must notify FDA of any authorized distributor(s) of your product, including the name, address, and phone number of any authorized distributor(s).
- J. You must provide authorized distributor(s) with a copy of this EUA and communicate to authorized distributor(s) any subsequent revisions that might be made to this EUA and its authorized accompanying materials, including the authorized labeling.
- K. You must make the authorized "iHealth COVID-19 Antigen Rapid Test Healthcare Provider Instructions for Use" and the Fact Sheet for Healthcare Providers electronically available on your website. Additionally, you must provide the opportunity to request a copy of the "iHealth COVID-19 Antigen Rapid Test Healthcare Provider Instructions for Use" and Fact Sheet for Healthcare Providers in paper form, and after such request, promptly provide the requested labeling at no additional cost.

- L. You may request changes to this EUA for your product, including to the Scope of Authorization (Section II in this letter) or to the authorized labeling, including requests to make available additional authorized labeling specific to an authorized distributor. Such additional labeling may use another name for the product but otherwise must be consistent with the authorized labeling and shall not exceed the terms of authorization of this letter. Any request for changes to this EUA should be submitted to DMD/OHT7-OIR/OPEQ/CDRH and require appropriate authorization from FDA prior to implementation.
- M. You must comply with the following requirements pursuant to FDA regulations: 21 CFR 820 Subpart H (Acceptance Activities, 21 CFR 820.80 and 21 CFR 820.86), Subpart I (Nonconforming Product, 21 CFR 820.90), and Subpart O (Statistical Techniques, 21 CFR 820.250).
- N. You must have lot release procedures and the lot release procedures, including the study design and statistical power, must ensure that the product released for distribution meet the clinical and analytical performance claimed in the authorized labeling.
- O. If requested by FDA, you must submit your lot release procedures to FDA, including sampling protocols, testing protocols, and acceptance criteria, that you use to release lots of your product for distribution in the U.S. If such lot release procedures are requested by FDA, you must provide them within 48 hours of the request.
- P. You must evaluate the analytical limit of detection and assess traceability<sup>11</sup> of your product with any FDA-recommended reference material(s). After submission to and concurrence with the data by FDA, you will update your labeling to reflect the additional testing. Such labeling updates will be made in consultation with, and require concurrence of, DMD/OHT7-OIR/OPEQ/CDRH.
- Q. You must evaluate the clinical performance of your product to support the serial screening claim in an FDA agreed upon post authorization clinical evaluation study within 6 months of the date of this letter (unless otherwise agreed to with DMD/OHT7-OIR/OPEQ/CDRH). After submission to and concurrence with the data by FDA, you must update the authorized labeling to reflect the additional testing. Such labeling updates will be made in consultation with, and require concurrence of, DMD/OHT7-OIR/OPEQ/CDRH.
- R. You must complete the agreed upon real-time stability study for your product and notify DMD/OHT7-OIR/OPEQ/CDRH of the testing results as they become available until completion of the study. After submission of the study data, and review and concurrence with the data by FDA, you must update your product labeling to reflect the additional testing if requested by FDA. Such labeling updates must be made in consultation with, and require concurrence of, DMD/OHT7-OIR/OPEQ/CDRH.

<sup>&</sup>lt;sup>11</sup> Traceability refers to tracing analytical sensitivity/reactivity back to an FDA-recommended reference material.

- S. You must complete your previously agreed upon automatic test reporting-related software updates to the iHealth COVID-19 Test App within 1.5 months of this letter and notify DMD/OHT7-OIR/OPEQ/CDRH upon implementation. Upon implementation, you must ensure automatic test result reporting is available, using the iHealth COVID-19 Test App, to relevant public health authorities in accordance with local, state, and federal requirements.
- T. You must submit to FDA a summary report within 90 calendar days of product launch summarizing the results of any testing performed using your product during that timeframe, including how many products were distributed, the positivity rate for specimens tested with your product, and how many individuals reported results to their healthcare provider as encouraged by the "iHealth COVID-19 Antigen Rapid Test Instruction for use," along with any proposed corrective action, as necessary.
- U. You must evaluate the impact of SARS-CoV-2 viral mutations on your product's performance. Such evaluations must occur on an ongoing basis and must include any additional data analysis that is requested by FDA in response to any performance concerns you or FDA identify during routine evaluation. Additionally, if requested by FDA, you must submit records of these evaluations for FDA review within 48 hours of the request. If your evaluation identifies viral mutations that affect the stated expected performance of your device, you must notify FDA immediately (via email: <a href="mailto:CDRH-EUA-Reporting@fda.hhs.gov">CDRH-EUA-Reporting@fda.hhs.gov</a>).
- V. If requested by FDA, you must update your labeling within 7 calendar days to include any additional labeling risk mitigations identified by FDA, such as those related to the impact of viral mutations on test performance. Such updates will be made in consultation with, and require concurrence of, DMD/OHT7-OIR/OPEQ/CDRH.
- W. You must submit your product for any FDA-recommended independent evaluation to confirm the performance characteristics of your test, if requested by FDA. After submission to and concurrence with the data by FDA, you will update your labeling to reflect the additional testing. Such labeling updates will be made in consultation with, and require concurrence of, DMD/OHT7-OIR/OPEQ/CDRH.

### **Conditions Related to Printed Materials, Advertising and Promotion**

- X. All descriptive printed matter, advertising and promotional materials relating to the use of your product shall be consistent with the authorized labeling, as well as the terms set forth in this EUA and meet the requirements set forth in section 502(a), (q)(1), and (r) of the Act, as applicable, and FDA implementing regulations.
- Y. No descriptive printed matter, advertising or promotional materials relating to the use of your product may represent or suggest that this test is safe or effective for the detection of SARS-CoV-2.
- Z. All descriptive printed matter, advertising and promotional materials relating to the use of your product shall clearly and conspicuously state that:

- This product has not been FDA cleared or approved, but has been authorized by FDA under an EUA;
- This product has been authorized only for the detection of proteins from SARS-CoV-2, not for any other viruses or pathogens; and,
- The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

The emergency use of your product as described in this letter of authorization must comply with the conditions and all other terms of this authorization.

# V. Duration of Authorization

This EUA will be effective until the declaration that circumstances exist justifying the authorization of the emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 is terminated under Section 564(b)(2) of the Act or the EUA is revoked under Section 564(g) of the Act.

Jacqueline A. O'Shaughnessy, Ph.D.
Acting Chief Scientist
Food and Drug Administration

Enclosure